




CLEANSSES
 MOISTENS
PolyMem
 FILLS
 ABSORBS
 more healing • less pain®

Ferris Mfg. Corp.

PolyMem Multifunctional Dressings

PolyMem® USER'S GUIDE

-Tunnels-

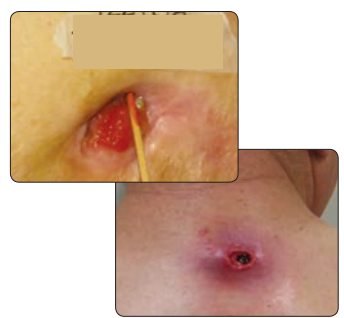


Cut cover dressing 1/4"-2" larger than the wound base.
 *Also available with antimicrobial silver

-Cavities-



Cut cover dressing 1/4"-2" larger than the wound base.
 *Also available with antimicrobial silver
 ** For use on moderate to heavy exudate or when additional wear time is desired

-Wounds less than 0.5cm depth-

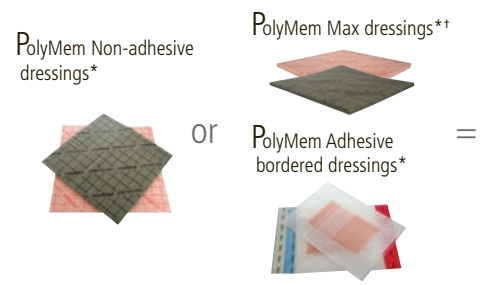

Cut cover dressing 1/4"-2" larger than the wound base.
 *Also available with antimicrobial silver
 ** For use on moderate to heavy exudate or when additional wear time is desired

EASY AS 1.2.3. POLYMEM USER'S GUIDE





Ferris Mfg. Corp.

PolyMem Multifunctional Dressings

PolyMem belongs to an innovative class of adaptable, multifunctional wound care dressings called polymeric membrane dressings. PolyMem dressings contain components that work together to promote rapid wound healing, pain relief and swelling reduction.

How can the characteristics of PolyMem dressings help you:

CLEANSSES: The built-in cleansing agent is activated by moisture and released into the wound bed, reducing the need for manual cleansing during dressing changes, which can interrupt the growth of healthy tissues.

FILLS: Gently expands and conforms to continuously fill the wound bed. The wound site will constantly receive the benefits of PolyMem dressings.

ABSORBS: Effectively wicks away up to 10 times its weight in exudate, allowing the natural growth factors and nutrients to concentrate in the wound bed.

MOISTENS: Helps keep the wound bed moist, preventing the dressing from sticking to the wound bed – dressing removal is virtually pain-free.

RELIEVES PAIN AND SWELLING: Both persistent and procedure-related wound pain are usually relieved by PolyMem's ability to continually moisten the wound bed. Polymem helps inhibit the nociceptor response, which helps reduce the swelling, pain and inflammation at and around the wound site.

PROTECTS FROM CONTAMINATION: Semi-permeable polyurethane film backing helps prevent contaminants from entering the dressing or wound bed.

Simple to Use.

Multiple configurations for a variety of wound needs.

EASY AS 1. 2. 3.

First Time Only:

- 1 Clean wound per facility protocol
- 2 Place PolyMem dressing on wound
- 3 Trace wound margin on top of dressing at each dressing change



If infection is present, treat appropriately

Dressing Changes:

- 1 Change when exudate reaches wound margin
- 2 Remove old dressing
NOTE: Do not disturb the wound bed
- 3 Place new PolyMem dressing on the wound

This is an overview. Please see package insert for complete instructions.

PolyMem dressings are:

- non-adherent
- available with and without silver
- not recommended for use in conjunction with topical treatments
- able to be left on the wound for up to 7 days, depending on exudate levels

Clinicians report superior results managing:

Skin Intact

- Pressure Ulcers (I)
- Suspected DTI
- Contusions
- Bruises
- Sprains
- Strains
- Trauma Injuries

Skin Broken

- Pressure Ulcers (II - IV)
- Leg Ulcers
- Diabetic Ulcers
- Burns
- Skin Tears
- Incisions
- Donor & Graft Sites
- Acute Wounds

Common dressings that PolyMem can replace:

- Polyurethane or silicone foams
- Hydrocolloids
- Hydrogels
- Transparent films
- Alginates
- Gauze and traditional dressings

Made in USA



Ferris Mfg. Corp. 5133 Northeast Parkway, Fort Worth, TX 76106 | Phone: 800.POLYMEM (765.9636) | International +1 630.887.9797 | Fax: 630.887.1008 | www.PolyMem.com

PolyMem, PolyMem Silver, PolyMem Wic, Wic, PolyMem Wic Silver, PolyMem Wic Silver Rope, PolyMem Max, Max, PolyMem Max Silver, Shapes, Shapes by PolyMem, The Shape of Healing, The Pink Dressing, SportsWrap, SportsWrapST, More Healing • Less Pain, Interlocking Circles Design, PolyMem For Sports, Not too Loose...Not too Tight...Just Right!, Ferris and FMC Ferris and design are marks owned by or licensed to Ferris. The marks may be registered or pending in the US Patent and Trademark Office and in other countries. Other marks are the property of their respective owners.

© 2012 Ferris Mfg. Corp.

MKL-498-B, Rev-2, 1212