# VERIFORTE™ med

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## **GB** Package leaflet: Information for the user

### 1. Range of application

For cleaning and moistening of wounds

### 2. How to use

For external use only.  
Suitable for repeated use under medical supervision.  
Veriforte™ med Wound Gel may be warmed to up to 37 °C before use.  
Clean the wound and surrounding area with Veriforte™ med Wound  
Irrigation Solution and dab dry. Apply Veriforte™ med Wound Gel to the  
affected areas and cover with a suitable secondary dressing.

### 3. Applications

Veriforte™ med Wound Gel may be used for the thorough wound  
cleaning of:

- acute or chronic wounds as well as surface wounds
  - cuts, abrasions and 1st and 2nd degree burns
  - critically colonised or infected wounds
  - cavities in the human body - such as the mouth, nose or ears
- Veriforte™ med Wound Gel may be used for moistening acute and  
chronic wounds as well as for keeping wounds moist.

### 4. Side effects

To date, there are no known side effects.

### 5. Contraindications

Do not use in the event of hypersensitivity or suspected allergy to the  
ingredients.

### 6. Interactions

Should not be combined with:  
• other wound cleaning products (such as enzymatic wound cleaners)

### 7. General safety instructions

Do not mix with other products or chemicals.  
Only use intact and undamaged packages.  
For external use only.  
Not for injection or infusion.  
Do not apply onto or inject into tissue using pressure.  
Observe expiry date.

Medical device – keep out of the reach of children!

### 8. Composition

Water, sodium chloride, hypochlorous acid, sodium hypochlorite,  
colloidal silicate

### 9. Technical information

Ready-to-use, clear gel.  
Once opened, to be used within 90 days.  
Expiry date: see label

### 10. Disposal

The empty packaging can be disposed of via the recycling collection.  
There are no specific disposal requirements.

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