Wundspüllösung  
Wound irrigation solution  
Solution de rinçage des plaies  
Soluzione detergente per ferite  
Výplachový roztok na rany

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**GB** Package leaflet: information for the user

Read the package leaflet carefully, as it contains important information on how to use this medical device. Veriforte™ med Wound Irrigation Solution is a non-prescription medical device. It is necessary that you follow the instructions in the package leaflet in order to achieve the best treatment results.

- Keep the package leaflet safe. You may need to read it again.
- If you need any further advice, please contact your doctor, wound therapist or pharmacist.
- You must consult a doctor should your symptoms worsen.
- Please contact your doctor or pharmacist if you experience any side effects.

**Contents:**

- |                            |                              |
|----------------------------|------------------------------|
| 1. Introduction            | 6. Side effects              |
| 2. Application description | 7. Interactions              |
| 3. Areas of application    | 8. Ingredients / composition |
| 4. Application notes       | 9. Technical specifications  |
| 5. Contraindications       | 10. Disposal                 |

Veriforte™ med Wound Irrigation Solution is a ready-to-use, wound irrigation solution for the cleaning, moistening and irrigation of acute, chronic and contaminated wounds as well as 1st and 2nd degree burns.

**1. Introduction**

Regular cleaning of wounds prevents excessive proliferation of pathogenic micro-organisms and thus prevents wound infection. In wounds that are already infected, adjuvant wound irrigation can help to significantly reduce the microbial load. This results from the mechanical effect of the irrigation. In addition, wound slough, necrotic tissue, cell debris and biofilm can delay or even prevent wound healing. Such obstacles can be removed through the excellent cleaning effect of Veriforte™ med Wound Irrigation Solution, and by creating a physiological wound environment, wound healing is further promoted.

Debris and contamination with micro-organisms cannot always be ruled out in the case of acute wounds. Thorough wound cleaning is to be strongly recommended in order to prevent infection.

**2. Application description**

Veriforte™ med Wound Irrigation Solution is suitable for the irrigation of acute, chronic and contaminated wounds and 1st and 2nd degree burns. In addition to this, Veriforte™ med Wound Irrigation Solution can be used for the irrigation of body cavities such as the mouth, nose, throat, ears and peritoneal cavity. Veriforte™ med Wound Irrigation Solution is suitable for the intraoperative irrigation of surgical sites. Veriforte™ med Wound Irrigation Solution is not suitable for systemic use, i.e. Veriforte™ med Wound Irrigation Solution must not be administered by intravenous or intra-arterial injections or infusions. Oral ingestion, although harmless, is nevertheless not recommended.

**3. Areas of application**

Veriforte™ med Wound Irrigation Solution can be used for the thorough cleaning of wounds such as:

- all chronic wounds of any depth, such as: diabetic foot ulcer, all forms of leg ulcer, pressure sores
- all acute wounds such as cuts, bites, lacerations, burst wounds and abrasions
- surgical wounds (intraoperative and postoperative)
- wounds with exposed cartilage, tendons, ligaments or bone
- 1st and 2nd degree burns
- radiation ulcers
- fistulas and abscesses
- body cavities such as the mouth, nose and ears
- critically colonized or infected wounds
- soft tissue injuries

Veriforte™ med Wound Irrigation Solution can be used for instillation in NPWT (negative pressure therapy). Veriforte™ med Wound Irrigation Solution can be used for moistening wound dressings and keeping wounds moist as well as for loosening bandages and wound dressings.

**4. Application notes**

Veriforte™ med Wound Irrigation Solution does not need to be rinsed or neutralised. The wound should be irrigated thoroughly with Veriforte™ med Wound Irrigation Solution. To enhance the cleaning effect, it is recommended that the solution be allowed to soak in for 60 seconds so as to dissolve any encrustations. This procedure can be repeated several times a day or at each dressing change. Alternatively, the wound may also be cleaned with gauze soaked in Veriforte™ med Wound Irrigation Solution.

Encrusted wound dressings are easily loosened with Veriforte™ med Wound Irrigation Solution and without trauma. To support the mechanical cleaning of the wound, it is useful to extend the application time of Veriforte™ med Wound Irrigation Solution to 15 minutes in order to also remove stubborn encrustations.

Veriforte™ med Wound Irrigation Solution can be used for up to 60 days once opened. The date on which it is first opened is to be written in the space provided on the bottle. Veriforte™ med Wound Irrigation Solution may be warmed to up to 37 °C before use.

**5. Contraindications**

Do not use in the event of hypersensitivity or suspected allergy to the ingredients.

**6. Side effects**

To date, there are no known side effects.

**7. Interactions**

Should not be combined with:  
• other wound cleaning products (such as enzymatic wound cleaners)

**General safety information**

- Do not mix with other products or chemicals
- Only use intact and undamaged packages
- Not suitable for systemic use (ingestion, injection and infusion)
- Observe the expiry date and batch number
- Medical device - Keep out of the reach of children
- Protect from direct sunlight

**8. Ingredients / composition**

Water, sodium chloride, hypochlorous acid, sodium hypochlorite

**9. Technical information**

Clear and colourless aqueous solution, which can have a slight smell of chlorine. The expiry date is stated on the label and must be observed. The solution can be used for 60 days once opened. The date on which it is first opened is to be written on the label on the bottle.

**9. Disposal**

The empty packaging can be disposed of via the recycling collection. There are no specific disposal requirements.

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